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# SYSTEM OF NARCOTICS IDENTIFICATION

## INSTRUCTIONS FOR USE

### TEST J

#### FOR: PCP (Phencyclidine)

##### Test Function

1. This reagent system presumptively identifies PCP.

##### How to Use Test J

1. Remove clip and insert into the test pack an amount of powdered suspect material that would fit inside this circle.  Reseal with clip and tap gently to assure material falls to bottom of pack.
2. With printed side of the test pack facing you, break ampoules from left to right. Break by squeezing the center of the ampoule with the tips of thumb and forefinger.
3. Break the left ampoule. Agitate gently for 30 seconds.
4. Break middle ampoule and agitate gently. Blue specks in a pink solution or solid blue indicates PCP.
5. Break right ampoule and agitate gently. If solid blue or blue specks remain, presume PCP is present.

##### Interpretation of Resulting Colors

1. The reagent will cause no color development (when the first ampoule is broken) in presence of PCP.
2. **IMPORTANT:** Lower grades of PCP will result in blue specks in a pink solution. Therefore, either blue or blue specks in a pink solution should be considered a positive test for the presumptive identification of PCP.
3. Illicit PCP is being found in a wide range of grades. With higher grades of PCP, an intense blue color will result. The pink color shown on the test packet and instruction manual may be totally obscured.
4. It is advisable that this test be used in conjunction with others in the Polytesting System.

#### CAUTION

No attempt should be made to crush glass particles after ampoules are broken.

Since these tests contain strong acid, it is suggested that Test F (Acid Neutralizer) be used after testing and before disposal. Before discarding used test packs, remove clip and add one measure of acid neutralizer from Test F. Add slowly to prevent bubbling over. Do not reseal the test pouch until effervescence has stopped completely. Reseal the test pouch with clip and discard in a tamper-free disposal unit.

**Antidote:** In case of contact, immediately flush eyes or skin with water for at least 15 minutes while removing contaminated clothing and shoes. Call a physician. If swallowed, do not give emetics. Contact a physician.

#### WARNING

These tests are NOT designed for use with liquid samples.

13386 International Parkway  
Jacksonville, Florida 32218  
904.485.1836 FAX: 904.741.5407

13386 International Parkway  
Jacksonville, Florida 32218  
904.485.1836 FAX: 904.741.5407

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